



PURPOSEFULLY
DESIGNED

Supris[®]
Retropubic Kit

FEMALE STRESS URINARY INCONTINENCE



PURPOSEFULLY PUT TOGETHER

The Supris® Retropubic Kit consists of the Supris implantable midurethral sling and disposable introducers that allow for either a suprapubic (“top-down”) or retropubic (“bottom-up”) surgical approach. It is indicated for the surgical treatment of female stress urinary incontinence (SUI) resulting from urethral hypermobility and/or intrinsic sphincter deficiency.¹

Dual-use
introducers

Distinct curve for use in
both a retropubic
and suprapubic
approach¹

Wide
surface
area

provides support as a
backboard for
the urethra¹

Predictable
tension

Mesh facilitates
positioning
during surgery¹

Macroporous
design

allows for optimal
tissue integration¹

Coloplast SUI Mesh: Purposefully Designed

Thin, lightweight mesh features low elasticity and wide surface area designed to provide support for the urethra.¹

Pore size

About 5x the minimum

Elasticity

Low: 7.5%

Thinness

About half as thin as others
on the market: ≤ .40 mm

Fiber size

Lower than others
on the market: .08 mm

Density

Lower than others
on the market: 70 g/m²

Total material/mass

Low amount of total
material implanted

Coloplast Delivers Confidence

It all started in 1954 with Elise Sorensen, a nurse who wanted to help her sister regain confidence and control. After having an ostomy operation, her sister was afraid to go out in public for fear of leakage. Elise worked with engineer Aage Louis-Hansen to develop the world's first adhesive ostomy bag for greater protection and control. Today, Coloplast is an environmentally conscious global leader in ostomy care, wound and skin care, and urology care products that improve lives for millions of people around the world.

To Order Call Toll Free 800.258.3476

This product may be ordered directly from Coloplast.

Product	Order Number
Supris® Retropubic Kit	93-4450

SUPRIS® RETROPUBIC KIT

Indications:

The Supris Retropubic Kit consists of the Supris implantable midurethral support sling and disposable introducers for placement using a "top-down" or "bottom-up" retropubic surgical approach. The Supris sling and introducers are indicated for the surgical treatment of female stress urinary incontinence (SUI), resulting from urethral hypermobility and/or intrinsic sphincter deficiency.

Contraindications:

It is the responsibility of the physician to advise the prospective patients or their representatives, prior to surgery, of the contraindications associated with the use of this product. The Supris Retropubic Kit is contraindicated for use in patients with the following conditions:

- Pregnancy or desire for future pregnancy
- Potential for further growth (e.g., adolescents)
- Known active urinary tract infection and/or infection in operative field
- Taking anti-coagulant therapy
- Abnormal urethra (e.g., fistula, diverticulum)
- Intraoperative urethral injury
- Any condition, including known or suspected pelvic pathology, which could compromise implant or implant placement
- Sensitivity/allergy to polypropylene

Warnings and Precautions:

It is the responsibility of the physician to advise the prospective patients or their representatives, prior to surgery, of the warnings and precautions associated with the use of this product and the associated surgical risks.

Warnings:

The Supris Retropubic Kit should only be used by physicians familiar with the surgical procedures and techniques involving transvaginal placement of non-absorbable, synthetic mesh slings and who have adequate education and experience in the treatment of female SUI.

A thorough assessment of each patient should be made to determine the suitability of a synthetic mesh sling procedure.

The patient should be counseled that alternative incontinence treatments may be appropriate, and the reason for choosing a mesh sling procedure should be explained.

Obtain patient consent prior to surgery and ensure that the patient has an understanding of the postoperative risks and potential complications of transvaginal mesh sling surgery.

Patient counseling should include a discussion that the sling to be implanted is a permanent implant and that some complications associated with the implanted mesh sling may require additional surgery; repeat surgery may not resolve these complications. Serious adverse tissue responses or infection may require removal of mesh, and complete removal of the sling may not always be possible. Individuals may have varying degrees of collagen laydown that may result in scarring.

As with all surgical procedures, patients with certain underlying conditions may be more susceptible to postoperative bleeding, impaired blood supply, compromised/delayed healing, or other complications and adverse events.

The risks and benefits of using Supris should be considered in patients.

Any future pregnancy could negate the benefits of this surgical procedure. Patients should report any bleeding, pain, abnormal vaginal discharge or sign of infection that occur at any time.

Do not use product that has damaged or opened packaging, or has expired, as sterility may be compromised.

The procedure to insert the Supris sling requires good knowledge of pelvic anatomy and the correct use of the introducer needles in order to avoid damage to adjacent anatomical structures.

Cystoscopy should be performed to confirm bladder and urethral integrity.

Avoid placing excessive tension on the Supris sling during placement and adjustment to maintain sling integrity and to avoid compression of the urethra when tensioning.

Precautions:

The Supris Sling and Supris introducers are provided sterile (ethylene oxide sterilization) and are for single-use only.

Use caution to prevent intraoperative injury to adjacent pelvic structures.

Do not let the Supris sling come into contact with sharp objects (e.g., staples, clips, or clamps) which could cause damage to the sling.

Potential Complications

Adverse events are known to occur with transvaginal synthetic sling procedures and implants. Adverse events following mesh implantation may be de novo, persistent, worsening, transient, or permanent.

Adverse events may include but are not limited to: abscess (acute or delayed), adhesion/scar formation, allergy, hypersensitivity or other immune reaction, bleeding, hemorrhage or hematoma, bowel obstruction, dehiscence, delayed wound healing, extrusion, erosion or exposure of mesh sling into the vagina or other structures or organs, fistula formation, infection, inflammation (acute or chronic), local irritation, necrosis, de novo and/or worsening dyspareunia, neuromuscular symptoms (acute or chronic), pain (acute or chronic), partner pain and/or discomfort during intercourse, perforation or injury of soft tissue (e.g., muscles, nerves, vessels), structures, or organs (e.g., bone, bladder, urethra, ureters, vagina), seroma, sling migration, bladder storage dysfunction (e.g., increased daytime frequency, urgency, nocturia, overactive bladder, urinary incontinence), ureteral obstruction, urinary tract infection, voiding symptoms (e.g., dysuria, urinary retention, incomplete emptying, straining, positional voiding, weak stream), granulation tissue formation, palpable mesh (patient and/or partner), sexual dysfunction, vaginal discharge (abnormal) and vaginal scarring or tightening.

The occurrence of these events may require one or more revision surgeries, including removal of the sling.

Complete removal of the sling may not always be possible, and additional surgeries may not always fully correct the complications.

There may be unresolved pain with or without mesh sling explantation.

The information provided is not comprehensive with regard to product risks. For a comprehensive listing of indications, contraindications, warnings, precautions, and adverse events refer to the product's Instructions for Use. Alternatively, you may contact a Coloplast representative at 1-800-258-3476 and/or visit the company website at www.coloplast.us.

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

References

1. Data on file at Coloplast.

Ostomy Care
Continence Care
Wound & Skin Care
Urology Care

Coloplast Corp. Minneapolis, MN 55411 / Urology Care Surgical Support 1-800-258-3476

www.coloplast.us The Coloplast logo is a registered trademark of Coloplast A/S.

© 2018 Coloplast Corp. All rights reserved.



PM-04687 06.18